


# The lancing device that works with a simple 1-Click action

## Prime and prick in 1-Click action



### Obtaining a blood sample

Prime and prick in 1-Click action.

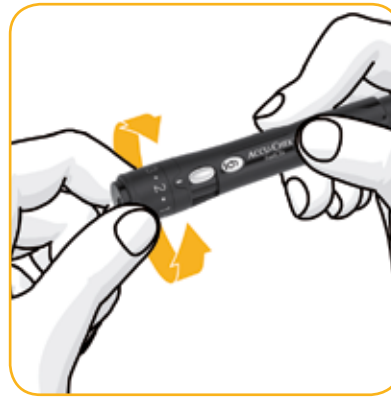

### Select penetration depth

Adjust the cap to set the correct penetration depth.





### Changing a lancet

Slide lever back and forth to change to a new lancet.




The Accu-Chek FastClix lancing device is intended for patient self-monitoring by an individual person only. It must not be used to collect blood in a multipatient setting.

# Accu-Chek® FastClix Replacing the lancet

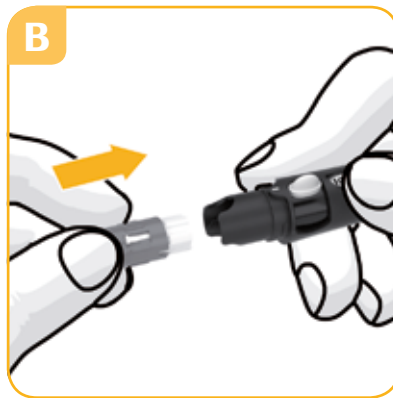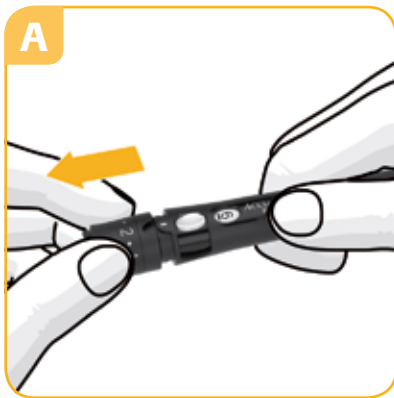


- A** Remove the cap.
- B** Insert the lancet drum.
- C** Replace the cap.

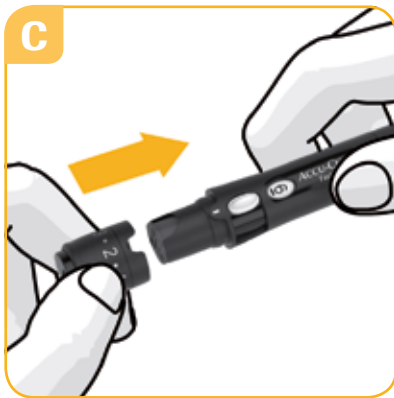

Replace the lancet drum after you have used the last lancet. Used lancet drums cannot be reused once removed.

## Accu-Chek® products

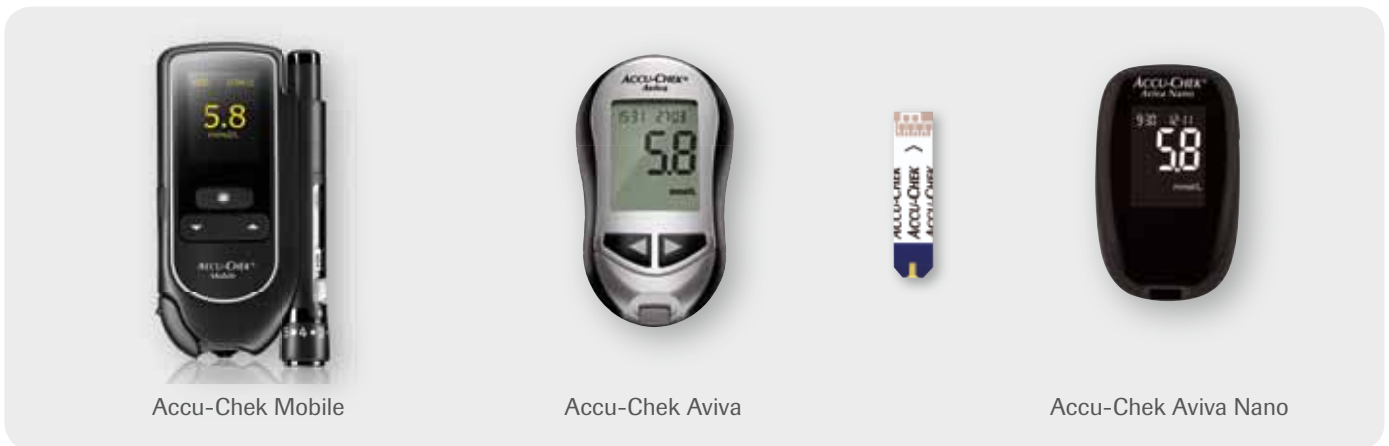

Accu-Chek Mobile

Accu-Chek Aviva

Accu-Chek Aviva Nano

Discover the new way to take a blood sample: your fingertips will thank you!

ACCUCHEK and FASTCLIX are trademarks of Roche. © 2012 Roche.

[www.accu-chek.nl](http://www.accu-chek.nl)  
0800-022 05 85  
Roche Diagnostics  
Postbus 1007  
1300 BA Almere



**ACCUCHEK®**